

MediCleanse™ Sterile Saline Wound Wash gently cleans and irrigates wounds, scrapes, and minor burns.

- **NO ALCOHOL**
- **NO PRESERVATIVES**
- **NO FRAGRANCES**

### **INGREDIENTS:**

Sterile 0.9% sodium chloride USP solution.

### **USES:**

To cleanse wounds, scrapes, and minor burns.

### **DIRECTIONS:**

- Before use, expel a short stream of Saline Wound Wash to clear nozzle.
- To dispense, aim nozzle and press. Apply spray to affected area.
- Flush area with Saline Wound Wash. Adjust distance as necessary.
- May be applied as often as needed or as directed by a healthcare professional.
- Replace cap after each use.

See bottom of can for lot number and expiration date.

### **WARNINGS:**

- **For external use only. Not for injection.**
- **For single-patient use only.**
- **Consult a health care professional before use for animal bites, deep puncture wounds and serious burns.**
- **Stop use and ask a healthcare professional if condition worsens or does not improve within 7 days.**
- Contents under pressure.
- Do not puncture or incinerate.
- **KEEP OUT OF REACH OF CHILDREN.**
- Store between 59° and 86°F (15° and 30°C); avoid excessive heat.

Distributed by:  
Ascent Consumer Products  
Melville, NY 11747 ©2022

